MicroLife NEB 100A / NEB 100B

EN RU UA KK RO

Distributor Europe / Middle-East / Africa MicroLife AG...

Asea MicroLife Corporation 9F, 431, RuiGang Road, NeiHu Taipei, 11492, Taiwan, R.O.C.

North / Central / South America MicroLife USA, Inc. 1617 Gulf to Bay Blvd., 2nd Floor Ste A Clearwater, FL 33755 USA

Compressor Nebuliser

- 1 Piston compressor 2 Power lead 3 ON/OFF Switch 4 Air filter compartment 5 Nebuliser 6 Air hose 7 Mouthpiece 8 Adult face mask 9 Child face mask 10 Replacing air filter 11 Assembling nebuliser kit 12 Nose piece

Dear Customer. This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.

Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product.

Read the instructions carefully before using this device. Type B applied part

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application. This device is not suitable for anaesthesia and lung ventilation. This device should only be used with original accessories as shown in these instructions. Do not use this device if you think it is damaged or notice anything unusual. Never open this device. This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section. Protect it from: water and moisture, extreme temperatures, impact and dropping, contamination and dust, direct sunlight, heat and cold. Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy. Never bend the nebuliser over 60°.

Ensure that children do not use this device unsupervised, some parts are small enough to be swallowed.

Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste. Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nebuliser kit (1). Ensure that all parts are complete. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level. Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (230V - 50Hz). Switch the ON/OFF switch (3) to position «I» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose. The mouthpiece gives you a better drug delivery to the lungs. Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely. Use all accessories including the nose piece (12) as prescribed by your doctor. Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. Do not lie down while inhaling. Stop inhalation if you feel unwell. After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «O» to turn off the device and unplug it from the socket. Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting». This device was designed for intermittent use of 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

3. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residues and possible impurities after each treatment. The compressor (1) and the air hose (6) should be cleaned with a clean, moist cloth.

- Always wash your hands well before cleaning and disinfecting the accessories. Do not expose the compressor to water or heat. Replace the air hose for each treatment with a new patient or in case of impurities. Unplug the device before cleaning.

Warning with water

- Wash all nebuliser components (except air hose) under warm tap water (max 60 °C) for about 5 minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer. Rinse thoroughly making sure that all deposits are washed away and leave to dry. All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). EN554/ISO11134. The sterilisation packaging must conform to EN868/ISO11607 and be suitable for steam sterilisation. After sterilization always let all components cool down to ambient temperature before further use. Do not repeat sterilization cycle when components are still warm.

4. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact MicroLife Service (see forward).

- We recommend replacing the nebuliser, mouthpiece, and the face masks after 3 months use. Check the filter continually for cleanliness and replace it if dirty, or after a maximum of 3 months use. Spare filters are provided with the device. Replace the filter, open the air filter compartment (4) on the top of the device marked «Filters» and exchange the filter (5). Never unscrew the plastic disc. The device cannot be switched on. Ensure the power lead (2) is correctly plugged into the socket. Ensure the ON/OFF switch (3) is in the position «I». The device has automatically switched off due to overheating. Wait until it has cooled down and try again.

- The nebuliser functions poorly or not at all. Ensure the air hose (6) is correctly connected at both ends. Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one. Ensure the nebuliser (5) is fully assembled and the colored impactor (13) is placed correctly. Ensure the required medication has been added.

6. Garantie

This device is covered by a 3 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers only the compressor. The replaceable components like nebuliser, masks, mouthpiece, air hose, and filters are not included. Opening or altering the device invalidates the guarantee. The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

7. Technical Specifications

Table with 2 columns: Specification and Value. Includes Nebulisation rate, Particle size, Compressor air flow, Acoustic noise level, Power source, Power lead length, Nebuliser capacity, Residual volume, Operating limits, Operating temperature, Storage temperature, Weight, Dimensions, Reference to standards, and Technical alterations reserved.

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